

MODEL K-9000

Dr. Cover[®]

N95 RESPIRATOR

N95 FFP2 MASK



Breathable Fabric Technology

Adjustable Nose Piece 3D Design

PFE \geq 95% BFE \geq 95%

5 Layers Dimensional Protection

Airborne Particle Respiration System

Dust Protection PM 2.5

Disinfected And Sterilized



www.drcover.org




At Dr. Cover we believe you shouldn't have to choose between your lifestyle and pollution protection. We believe that your every breath you breathe should be pure.

So we've decided and developed an extensive anti - pollution filtration system using cutting edge science and technology. The result? Outstanding Pollution Protection fit for your Lifestyle.

Dr. Cover presents the K-9000 N95 Protect Anti-Pollution Filtration Mask, that provides 98% 5-layer filtration of bacteria, viruses, toxins, pollens, and PM 2.5 pollution particles.

Dr. Cover's K-9000 Anti-Pollution Filtration respirator has an adjustable nose clip which offers maximum comfort, and is designed to provide you with reliable respiratory protection. An ideal choice against dust, odor, pollens, and toxins. It features special filters that have an extended dust holding capacity, ensuring that the mask has a longer usable life, whilst also filtering out pollens and toxins to the smallest size, which may cause allergies and even an influenza virus suspended in the air. The adjustable edge 3D Face-Fit isolation technology helps the mask fit the face properly, eliminating any chance of leakage as well as allowing easy breathing and talking even while wearing the mask. Unlike a regular mask with elastic strings, for comfortable use, this disposable mouth respirator has stretchable straps that can gently be placed around the ear, without experiencing pain. The K-9000 respirator; certified by International test standards, provides additional protection from dust and liquid droplets in the air. It is ideal for all, men, women, and children.

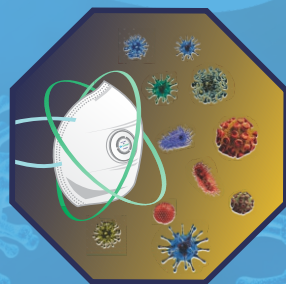



The astonishing technology, comfort, safety and breathability are not the only things that make this mask special.

Dr. Cover provides a UV-Light sterilisation standard for all Personal Protective Equipment (PPE) that comes out to you. This of course, includes the N95 Respiratory Protective Masks. This ensures that there are no chances of any bacteria or viruses to be in contact with you.

Dr. Cover understands what it means to be on the frontline fighting and risking your lives on a daily basis, and puts safety above all else.

We appreciate everyone who is helping the world during a difficult time like this, and we strive to provide great protection and comfort for everyone.



The Dr. Cover anti-pollution filtration masks use 5 Layer Media filtration technology which blocks even the smallest of particles responsible for air pollution.

These masks are made of High quality filters which blocks PM 2.5 dust, haze, bacteria, influenza viruses etc. The 5 layer filtration system increases effective filtration time by 50 percent, compared to similar products. These masks have a filtration capacity ranging from 95% to 98% depending upon the model being used.

In the 5 layer filter technology, the most crucial is the middle layer which is crafted with unique filtration technology; that's quite a rare sight for masks that won't be a burden on your pockets! This layer is responsible for filtering out up to 0.3 micron level of dust particles.

Always insist on masks which are approved to recognized standards such as BIS/EN/NIOSH/KN as a complete mask and do not get misguided by claims stating that the mask is made from materials conforming to these standards. A mask is always tested and approved as a complete with the accessories like headband, nose clip, filter valve, etc.

1. This anti-pollution filtration mask provides over 95% filtration of bacteria, as certified with international standards.
2. It is made of ultra-soft fabric that's gentle on skin and provides comfortable-fit.
3. It has adjustable straps with a soft nose pad. The mask is made from Face-fit 3D scanning techniques and technology, to fit your face with highest comfort.
4. This anti-pollution filtration mask is fit for the entire family. It is designed and tested to fit Children.

THINGS TO CHECK

WHEN YOU WEAR YOUR N95 RESPIRATOR:

After the respirator is on and formed to your face, check the fit of the respirator.

Place both hands over respirator.

Exhale sharply.

If air-leaks are detected, re-adjust respirator; exhale sharply again.

If air leaks are present, readjust the nosepiece, straps and secure the respirator to your face; check the fit again.

The respirator will not work if air leaks around the sides.

HOW TO REMOVE

YOUR N95 RESPIRATOR AFTER USE

Take the respirator off when you have completed your task.

Make sure to not touch anything while your hands have touched the outside of the respirator.

Clean your hands after removing the respirator.

Hang it to dry in a clean area so it will not be crushed.

Disposable N95 respirators can not be reused, and will have to be disposed of when the filter is all used up. You will not be able to get a tight seal when you reuse the mask.

Disposable respirators cannot be cleaned by washing, they must be disposed of when you can no longer get a tight seal.

Used respirators can be disposed of in the regular trash.

GENERAL PRECAUTIONS

N95 RESPIRATOR

People with chronic respiratory, cardiac, or other medical conditions that make breathing difficult should check with their health care provider before using an N95 respirator because the N95 respirator can make it more difficult for the wearer to breathe. Some models have exhalation valves that can make breathing out easier and help reduce heat build-up. Note that N95 respirators with exhalation valves should not be used when sterile conditions are needed

* It removes heat, moisture and carbon dioxide (Co2). And in combination with Dr. Cover K-9000 N95 Respirator, the Anti-Pollution Filtration Mask, you also get a 98% 5-layer filtration of pollution particles PM 2.5.

* All of Dr. Cover masks and PPE components are made with UV Light sterilisation as part of the production, ensuring no chances of any sort of bacteria and virus to be in contact with you.

Manufactured By
Dr. Cover (Venture Of Grandway Incorporated)

Grandway Incorporated, G T Road, Near Jalandhar
Bypass Ludhiana, Punjab 141008, India

www.grandwayinc.com

www.drcover.org